# ASSEMBLY INSTRUCTIONS

## Deluxe Folding Walker

- NL *Opvouwbaar looprek deluxe*
- FR *Cadre de marche pliable Deluxe*
- DE *Deluxe zusammenklappbare Gehhilfe*
- ES *Caminador plegable de lujo*
- PT *Andarilho dobrável deluxe*

**PR30180-SL**

*Aids to daily living*

**Able2**

Please inspect all parts carefully for cracks/breaks prior to use.


This walker comes fully assembled.

#### Dimensions

Height adjustable: 78 - 96 cm  
Width: 50 cm  
Depth: 40 cm  
Folded: 10 cm

Maximum user weight 120 kg.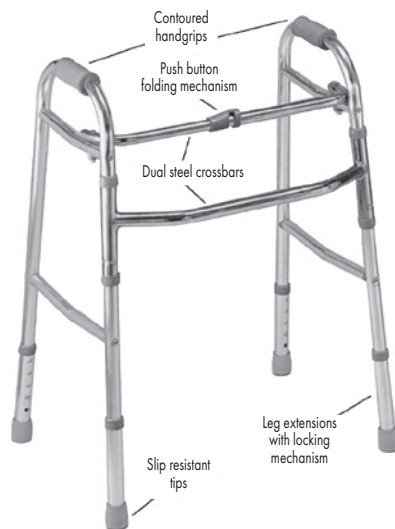

- When adjusting the height of the leg extension, make sure that each is set into the same height adjustment hole and that the walker is level. Snap buttons must fully protrude through the adjustment hole of each leg extension to ensure a positive lock.
- Persons with severe disabilities or persons who are unable to walk without an aide should not attempt to use this product without assistance.

#### USER INSTRUCTIONS

This walker is designed to help you balance or unload weight from your lower extremities while you walk.

**To open** rotate legs outward until legs lock securely in place.

**To close** engage folding mechanism by pushing down on the push button on the centre of the crossbar and rotating legs inward until fully closed.

To adjust handle height: stand up straight with arms at your side. Adjust the legs so the handgrips are level with the bend of the wrist. Elbows should be slightly bend when in use, to offer control and more stability.

Make sure that the legs of the walker are level to the ground. Place the walker at a comfortable distance in front of you. Using your arms for support, walk up into the frame of the walker. Do not step up completely against the front bar of the walker. This could throw you off balance. Repeat this process to continue walking.

When turning, take small steps so you won't lose your balance.

When sitting, back up all the way to the surface you are going to sit on until your legs touch the seating surface. Reach back with one hand to the seating surface to steady yourself as you sit. When rising from a seated position, push from the seating surface to rise, then place hands on the grips of the walker.

#### MAINTENANCE INSTRUCTIONS

Before use, check to make sure that the rubber tips are not ripped, worn, or missing. Replace if necessary before using.

#### CARE INSTRUCTIONS

Clean with mild cleanser and water; dry thoroughly.

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Valentine International Ltd. Taiwan
Able2 Europe B.V.
Able2 UK Ltd.