ENGLISH

Read carefully these instructions before the patient uses this raised toilet seat Muralla and keep them for future reference.

The raised toilet seat Muralla is designed for the sitting on the toilet seat and should never be used for any other use different to the intended one. Do not use as a step to climb on.

You should check regularly and before its use that the fixing elements are firmly secured.

Height adjustable: 5 cm, 9 cm, 12,5 cm.

Maximum patient weight: 150 kg.

ASSEMBLY

1. Take parts out of packaging.
2. Take the arm and locate it in the hole in the side of the toilet base at the rear, slide all the way until it reaches the spring loaded plunger, press the plunger in and slide the arm in further until you hear a click (Photo 1).
3. To adjust the height, undo the four black knobs on the seat and pull out the four white plugs for the height required, move the bars down or up depending on height required and screw in the knobs, replace the four white plugs in the holes.
4. To fit the seat to the toilet, remove the existing seat and lid, undo the two fixing screws on the seat and locate in the toilet, screw in the pan fixing screws to securely fasten your Muralla seat to the toilet (Photo 2).

CARE INSTRUCTIONS AND MAINTENANCE

Remove the dirtiness with a wet cloth and neutral soap and then dry. Do not use aggressive or abrasive chemical cleaners as they can damage the product.

At the end of its lifetime, get rid of the product in a responsive way.

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