MANUAL DE INSTRUCCIONES

Silla de ruedas

S220 Sevilla | S230 Sevilla

- Manual de instrucciones **ES**
- Instruction manual **EN**
- Manuel D'Utilisation **FR**
- Manuale D'Istruzioni **IT**
- Anweisungen **DE**
- Instruções **PT**
- Gebruiksaanwijzing **NL**
- Bruksanvisning **SWE**
- Instrukcja obsługi **PL**
- Brugsanvisning **DK**

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INDEX

1. SIMPLE STRUCTURE FIGURE
2. GENERAL SAFETY INSTRUCTIONS
3. DELIVERY AND PREPARATION FOR USE
4. TRANSPORTATION
5. ARMREST USAGE
6. FOOTRESTS USAGE
7. REPLACING THE SEAT UPHOLSTERY
8. REPLACING FRONT WHEELS OR FRONT WHEELS UNIT
9. ADJUSTING THE BRAKES
10. MAINTENANCE, CLEANING AND DISINFECTION
 - 10.1. MAINTENANCE
 - 10.2. CLEANING AND DISINFECTION
11. RE-USE

S220 SEVILLA: Self-Propelled wheelchair with 24" rear wheel
S230 SEVILLA: Transport wheelchair with 12" rear wheel

NOTICE

Notice to the user and/or patient: any serious incident that has occurred in relation to the device must be reported to the manufacturer and to the competent authority of the Member State where the user and/or patient is established.


Indications

This product is intended for disabled or elderly people who have difficulty moving.

Contraindications

Not recommended for persons who are too weak to sit up.

1. SIMPLE STRUCTURE FIGURE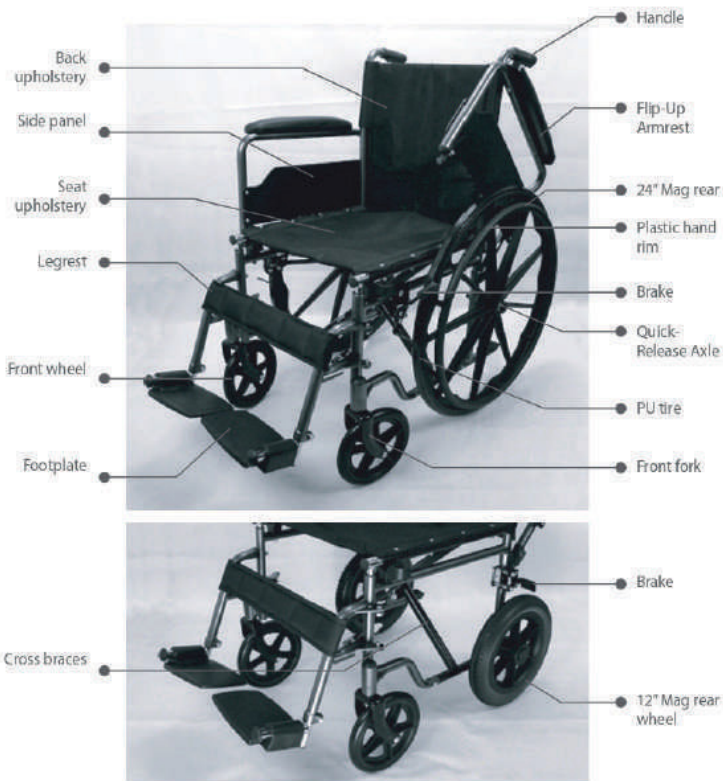

(Please Note: Following pictures and wheelchair you received may be differente, please in kind prevail.)

2. GENERAL SAFETY INSTRUCTIONS

2.1. Please read the instructions for use first. Before using the product, you should be familiar with the handling, function and use of the product. You may compromise your safety if you do not observe these instructions. However, it is impossible to protect against and avoid all unforeseeable circumstances and situations.

2.2. Do not ascend or descend stairs without the assistance of attendants. If devices such as ramps or elevators, they should be used. If wheelchair-friendly access is missing, two attendants must carry the wheelchair over the obstacle.

2.3. If possible, the footplates should be flipped up before getting into or out of the wheelchair.

2.4. Risk of accident when driving without experience. This can lead to falls and other dangerous situations. Therefore, familiarize yourself with the new wheelchair by practicing on even, straightforward terrain first. Learn how the wheelchair reacts when the center of gravity shifts; for example, on slopes or inclines or when clearing obstacles such as steps and curbs. This should be done only with the assistance of another person. Using an anti-tipper is strongly recommended for inexperienced wheelchair users.

2.5. All brakes acting on the tires do not serve as service brake but are only designed as parking brake (wheel lock). The wheel locks must not be used as driving brakes for slowing down the wheelchair, as in extreme cases, the abrupt stopping of the wheelchair can lead to falls.

2.6. Risk of damage caused by overload. The maximum load capacity is 120 kg.

2.7. The drive wheels must be replaced in case of tire damage (tread becomes worn down to such an extent that there is only 5 mm of radius left on the outer edges of the tire) or damage of the rim. The 12" drive wheels of the transfer version is replaced in the same way.

2.8. These standard wheelchairs are not approved for transportation in vehicle for the disabled. Wheelchair users must always transfer for the seats installed in the motor vehicle, non-observance of these instructions could lead to injury of the user, as well as of all other passengers in the motor vehicle in the event of an accident.


3. DELIVERY AND PREPARATION FOR USE

Your wheelchair is usually delivered fully assembled, folded, and with the rear wheels mounted. A few simple steps are enough to prepare it for use. The rear wheels of Models S-20 Sevilla and Model S230 Sevilla were unassembled, its wheels and axles are labeled with a letter R (Right) and L (Left) for a proper assembly. Please refer step 3.1. to assemble firstly before use.

The content in the packaging carton box is as follows:

1. Folded wheelchair.
2. Disassembled rear wheels (Models S220 Sevilla and S230 Sevilla) attached to the wheel structure.
3. Accessory box:
 - Footrest
 - Heel strap
 - Pre-Adjusted Quick-Release axles identified with R (Right) and L (Left).
 - Wrenches (Allen or hexagonal depending on the wheelchair model)
 - Instructions for use

3.1. Install rear wheels by Quick-Release axles. (Figure 1)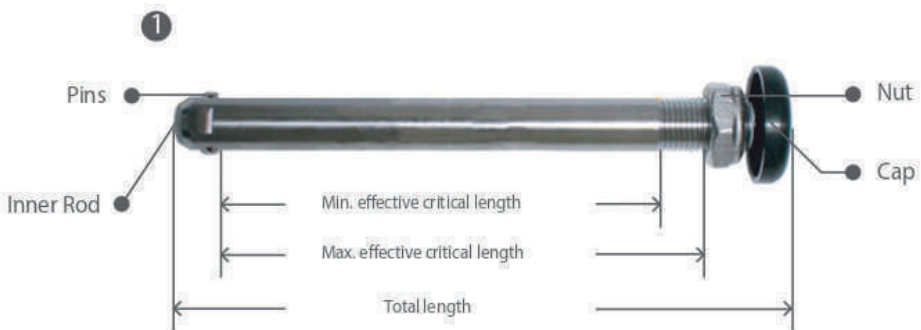

Press and hold the cap of axle, insert the center hole of rear wheel, and then insert the hole of wheelchair frame. (Figure 2).

Install the left axle and left rear wheel to the left side of the chair and install the right axle and right rear wheel to the right side of the chair. To determine about the sides of the wheelchair, unfold the chair and stand behind it, holding the handles: Right and left side will be your right and left hands subsequently.


Confirm the pins is complete upspring and locked the axle and rear wheels; if the pins are not complete upspring, the inner rod is protruding at the same time, this is a sign that the axle needs to be adjusted. A wrench will be needed to move the nut while keeping the axle in a fixed position.

Please, adjust the nut of axle to increase the effective critical length from pins to nut, until the pins are complete upspring and the inner rod is inside the axle. Please, note to check the axles frequently during use, if find the pins can't be upspring and/or the axles can't work, must replace it at once and please don't use wheelchair when the axles are broken or defective.


Warning: Replacement axles are not adjusted to your chair. They must be adapted following the steps in this paragraph for a proper assembly of the wheels. Please observe the side (R, L) during adaption.

Please note, the critical length couldn't be longer too much than wigth of E plus F (Figure item E, F), otherwise will cause the rear wheels shaking.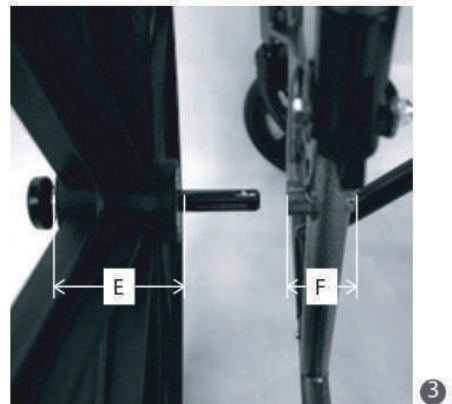

Warning: Before use wheelchair, please do confirm the rear wheels were fully inserted and settled, otherwise maybe cause the rear wheels detaching and personal injury and please adjust as above method.

3.2. While standing next to the wheelchair (Figura 1), unfold the wheelchair using flat hands (Figure 2). Ensure that the telescoping bars (Figure 1 item A) are completely collapsed and that the seat upholstery is flattened out.

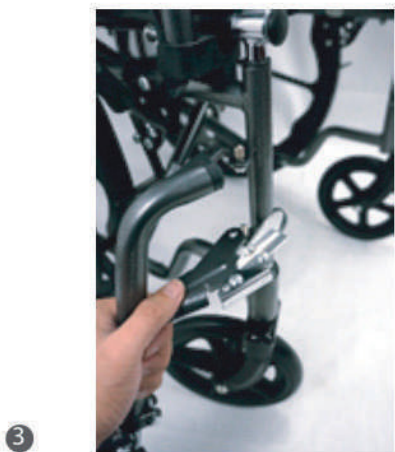3.3. If applicable, insert the footrests into the receivers from the side (Figure 3). Turn the footrests inward until locking to the right place. Finally, fold down the footplates after getting into wheelchair (Figure 4). if user wants to leave from wheelchair or stand up, please fold the footplates firstly.

4. TRANSPORTATION

Grab the middle of seat upholstery at the front and the back and pull it upwards, the wheelchair will be folded (Figure 5).

For more manageable transportations, e.g. in cars, the footrests can be entirely removed in just a few easy steps. To move the footrest, press the release (Figure 6 item B).

The footrest with footplate can then be swung outward by 90° and be removed by pulling it upward.

5

B

6

5. ARMREST USAGE

The side panels can be flipped-up to the rear for transfer user (Figure 8). To raise the side panel, manipulate the release lever (Figure 7) behind the side panel tube. The height of the armrest is not adjustable.

7

8

6. FOOTRESTS USAGE

The height of standard footrests can be changed with tools (Figure 9). To release the screw and then adjust the right place for use, finally tighten the screws.

7. REPLACING THE SEAT UPHOLSTERY


The seat upholstery can be replaced if necessary by loosening the star bolts. (Figure 10).

8. REPLACING FRONT WHEELS OR FRONT WHEELS UNIT

8.1. If necessary, the front wheels can be removed /installed with an open-end wrench/ ring wrench and an Allen wrench (Figure eitem A). For reassembly, please proceed in reverse order. Firmly retighten the screw connection.

8.2. If necessary, the front wheels unit can be removed/installed with an open-end wrench/ring wrench. Loosen and remove the nut above the caster mounting (Figure E). Pull the fork with the axle out of the bearings. Replace the front wheels unit. For reassembly, please proceed in reverse order. Firmly retighten the screw connection.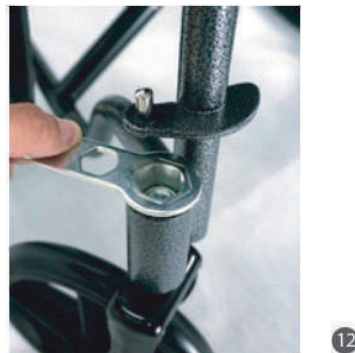

9. ADJUSTING THE BRAKES

The brakes are effective with proper adjustment (approx. 22mm distance, subject to technical changes). If necessary, the wheel lock can be set so that when the brake is not engaged the clearance between the tyre and brake block is approx. 22mm (Figure 13, 14, 15).

10. MAINTENANCE, CLEANING AND DISINFECTION

The proper function of the wheelchair, especially of the wheel locks, quick-release axles and brakes should be checked before every use. Safety nuts should be used only once. If they have been loosened several times, these nuts must be replaced.

Should any problems or wear be encountered, please contact your authorized specialist dealer immediately in order to have them corrected.

10.1. MAINTENANCE

CHECK	BEFORE EACH USE	MONTHLY	QUARTERLY
Function test of the wheel locks	X		
Sagging of the seat and back upholstery		X	
Check stability of footplate		X	
Visual inspection of wear parts (E.g., tyres, bearings)		X	
Dirt on bearings		X	
Push rings for damages		X	
Folding mechanism for wear and tear		X	
Spoke tension of the rear wheels			X
Screw connections			X

This is particularly important if you notice that the driving characteristics change, or the wheelchair becomes less stable. We also recommend that you have the wheelchair maintained regularly at 12 month intervals by your authorized specialist dealer. The functions described in the following list must be verified by the user at the specified intervals. To ensure smooth operation always, the following precautions and cleanings should be carried out. Users may carry these out themselves:

- Hair and dirt particles generally accumulate between the caster wheel and fork. This can restrict the caster wheels from rotating smoothly. Remove the casters and thoroughly clean the forks and casters using a mild household cleaner.

- If your wheelchair gets wet, towel dry it as soon as possible.
- Do not use the wheelchair in salt water and keep sand or other particles from damaging the wheel bearings.
- Screw connections should be checked frequently, when beginning to use the wheelchair and after any adjustment. If a screw connection comes loose repeatedly, please consult your specialist dealer.

10.2. CLEANING AND DISINFECTION

- Clean the upholstery with warm water and a washing-up liquid. Remove spots with a sponge or a soft brush. Rinse with clear water and let the components dry.
- Do not use any aggressive cleansing agents, solvents or hard brushes, etc.
- No wet-cleaning. Components are not machine washable. For disinfection, water-based agents should be used. The manufacturer's instructions for use must be observed.
- Before disinfecting, the pads and handles must be cleaned. Plastic parts, frame parts as well as chassis and wheels can be cleaned with a mild detergent. Dry thoroughly afterwards.

11. RE-USE

The product is suitable for re-use. Features and functions must not change in a way that could endanger patients or other persons within the product's life cycle.

The product can be used for a period of 4 years, if it is used properly and that the service and maintenance instructions are observed. It should be clearly pointed out, however, that the product is reliable far beyond this defined period, if it is cared for and maintained appropriately.

In cases of re-use, the affected product must first be thoroughly cleaned and disinfected. Subsequently, the product must be examined by an authorized specialist to check the conditions, wear and possible damage. Any worn and damaged components as well as components which do not fit or are not suitable for the user must be replaced.

If a product is to be disposed of, all components of the product must be disposed of properly in accordance with the applicable environmental protection regulations in the respective country.

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